

Radica Diagnostic Procedures

Radica Resin can be used as an esthetic, diagnostic display, as well as a provisional. Either way you choose to use the material, it must be processed identically. You must cure all restorations, diagnostic or provisional, before delivery to your accounts.



Working model.




Make 1.0 mm depth cuts.



Uniform 1.0 mm reduction.



Wax to full contour.

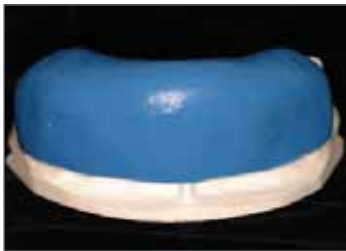


Make a silicon matrix.



Model Release.



Enamel.

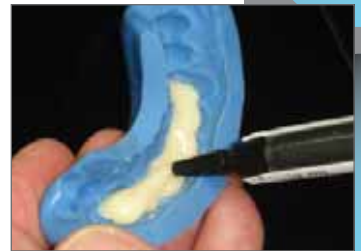

Dentin.



Matrix in place.



Trimming.



Sealer.



Curing.



Completed
Diagnostic/Provisional.

TABLE 1. CURE TIMES FOR DENTSPLY CURING UNITS

	EnterraTM Curing Unit	Triad[®] 2000 Curing Unit	Eclipse[®] Processing Unit
Initial cure	5 min	10 min	3 min
Pontic cure	1.5 min	2 min	1/2 min
VLC Sealer	2 min	2 min	1 min

Note: Do not use early versions of the Triad Curing Unit.